
# **5 Adjusting the Handle Height**

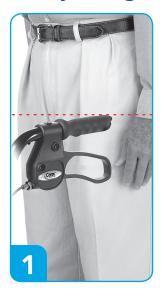





#### Using the brakes



Pull up to apply brakes









Push down to lock brakes



Carex Health Brands
PO Box 2526, Sioux Falls, SD 57101-2526
Phone 800.328.2935 www.carex.com





# Steel Rolling Walker



## 1 Check Parts





#### Rolling walker frame





Handles with brake assembly and back wheels x2



Front wheels



E-clips & hand screws with washers



Note: actual items may vary slightly from what is shown.

# 2 Handle Assembly



Cable should be outside frame.



NOTE: Hex shaped bolt should fit onto hex shaped hole in tubing. Hand screw faces out.

## 3 Front & Rear Wheel Assembly



Insert back and front wheels using E-Clip and hand screw to secure them into place.

Make sure all knobs are firmly tightened before use.





# 4 Backrest Assembly & Braking System



Push in side buttons and insert backrest. Flip up backrest for compact storage.

